

Instrument Moisturizing Gel 3060


- Moisturize persistently
- Efficiency for decontamination
- Excellent compatibility with equipment/machinery
- Bactericidal and bacteriostatic

Safer Supply, More Than Cleaner



Instrument Moisturizing Gel

Moisturizing performance test

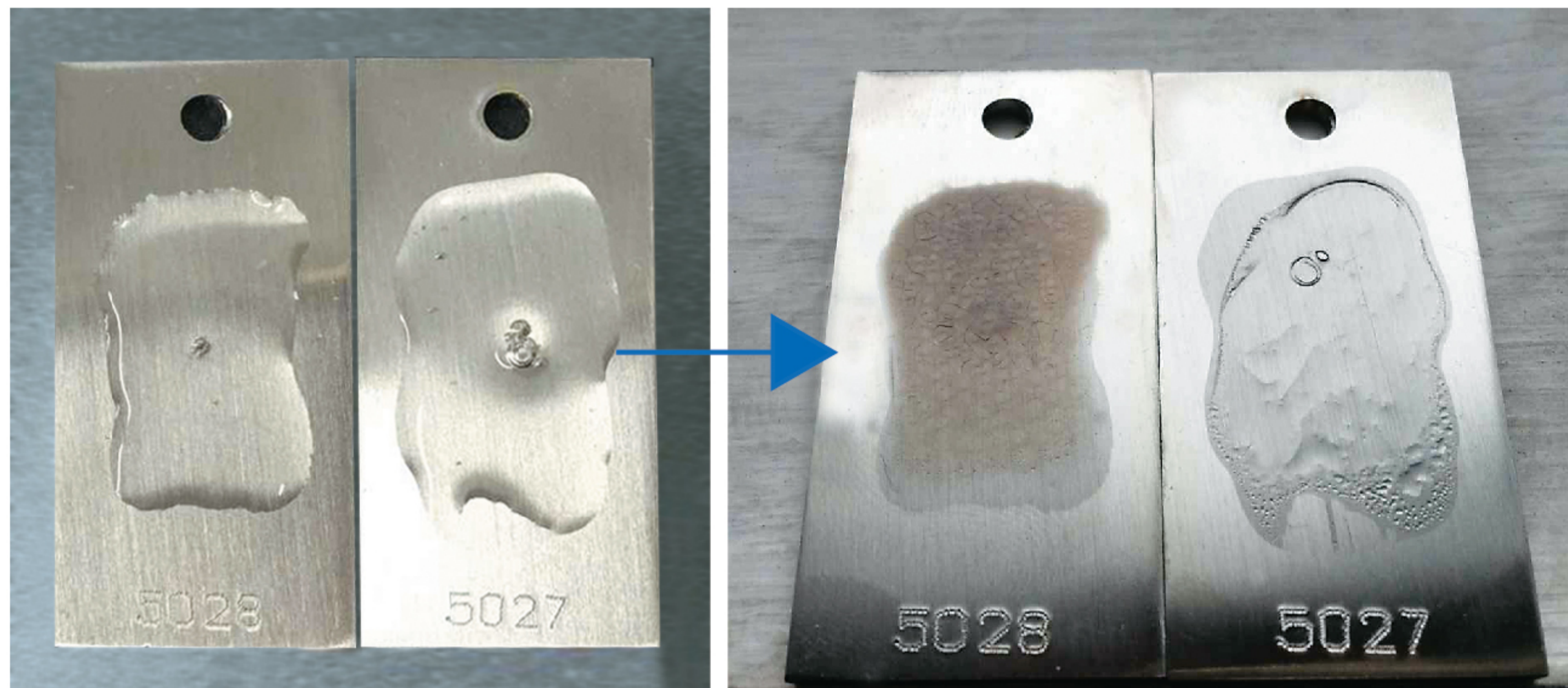

Product A in the market Conbizyme®

Product A in the market Conbizyme®

Drop the product of the same weight on the treated metal test blocks and keep for 72h in the natural state.

From the figure, the surface treated with the moisturizing agent A the same condition is uneven and cracked after 72h. But the test block treated with Conbizyme® instrument moisturizing gel is still in the uniform wet state.

Corrosion prevention performance test



Product A in the market Conbizyme®

Product A in the market Conbizyme®

Product A in the market Conbizyme®

After immersing the lower half of the metal piece in the humectant stock solution for 72 hours, it exhibited excellent rust prevention performance.

Operating instructions and packing specification

Recommendation: Conbizyme® Instrument moisturizer is a ready-to-ues product. It could be sprayed directly about 10cm away from the dirt to ensure all the dirt is covered on average with no dilution. You can choose to rinse before cleaning if necessary according to the residue of

Packing specification

Product name	Model	Packing specification
Conbizyme® Instrument Moisturizing Gel	3060	500mL (10 bottles per box)