

Rust Remover

3055


- Remove rust Effectively
- Suppress device re-corrosion
- Excellent device/equipment compatibility

Safer Supply, More Than Cleaner



Rust Remover


Rust removal performance comparison test

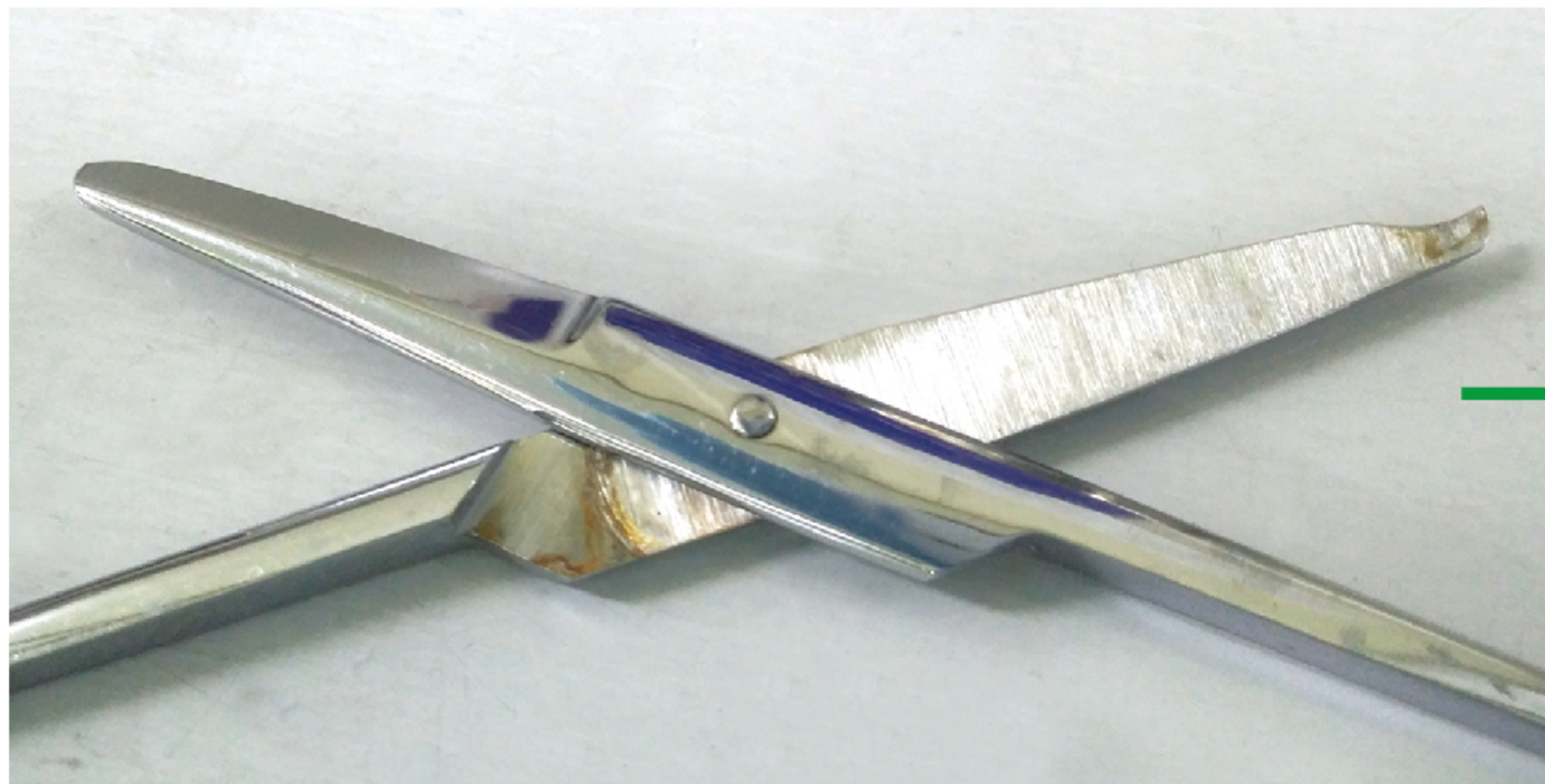


Before treatment



After treatment

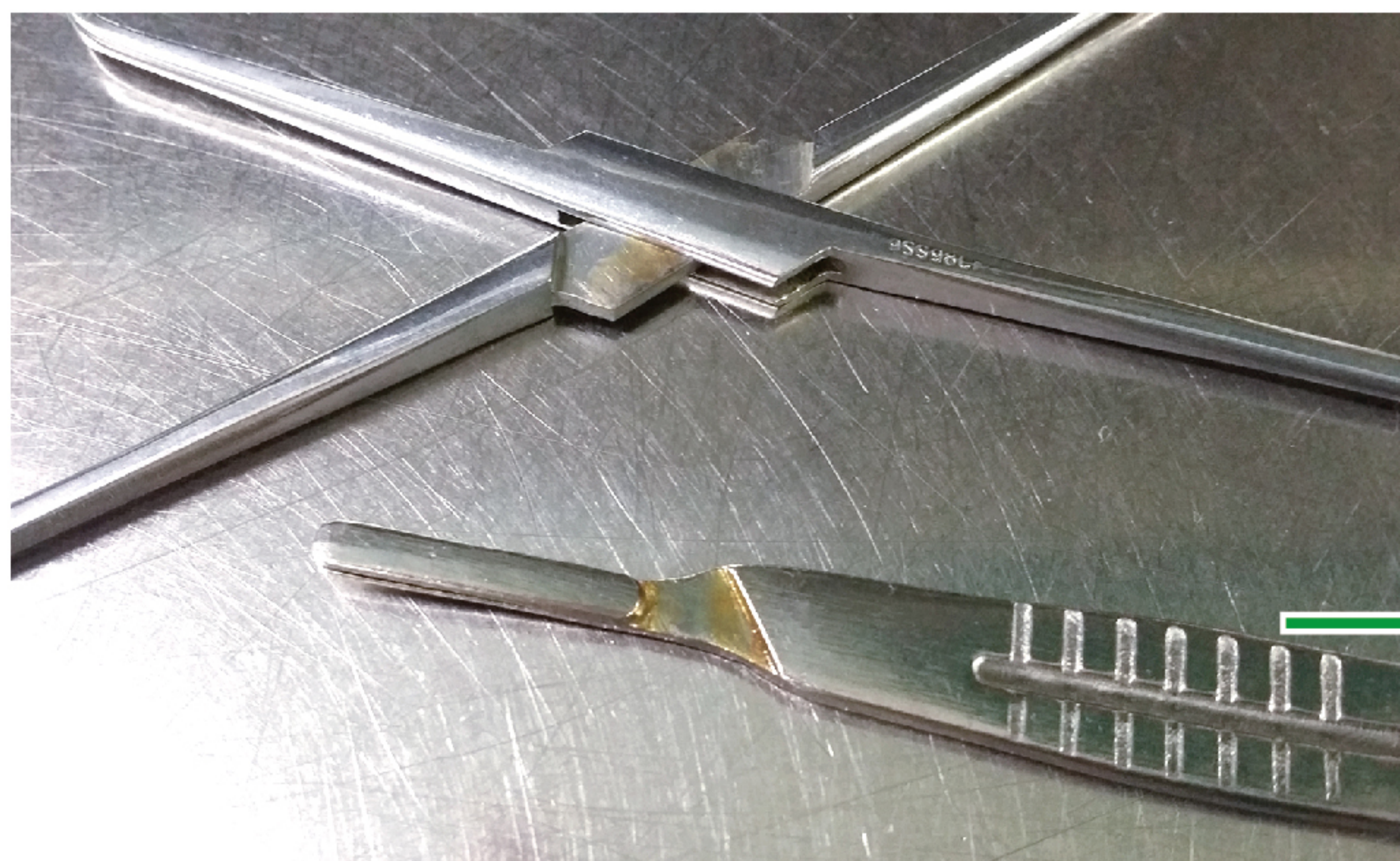

Before treatment

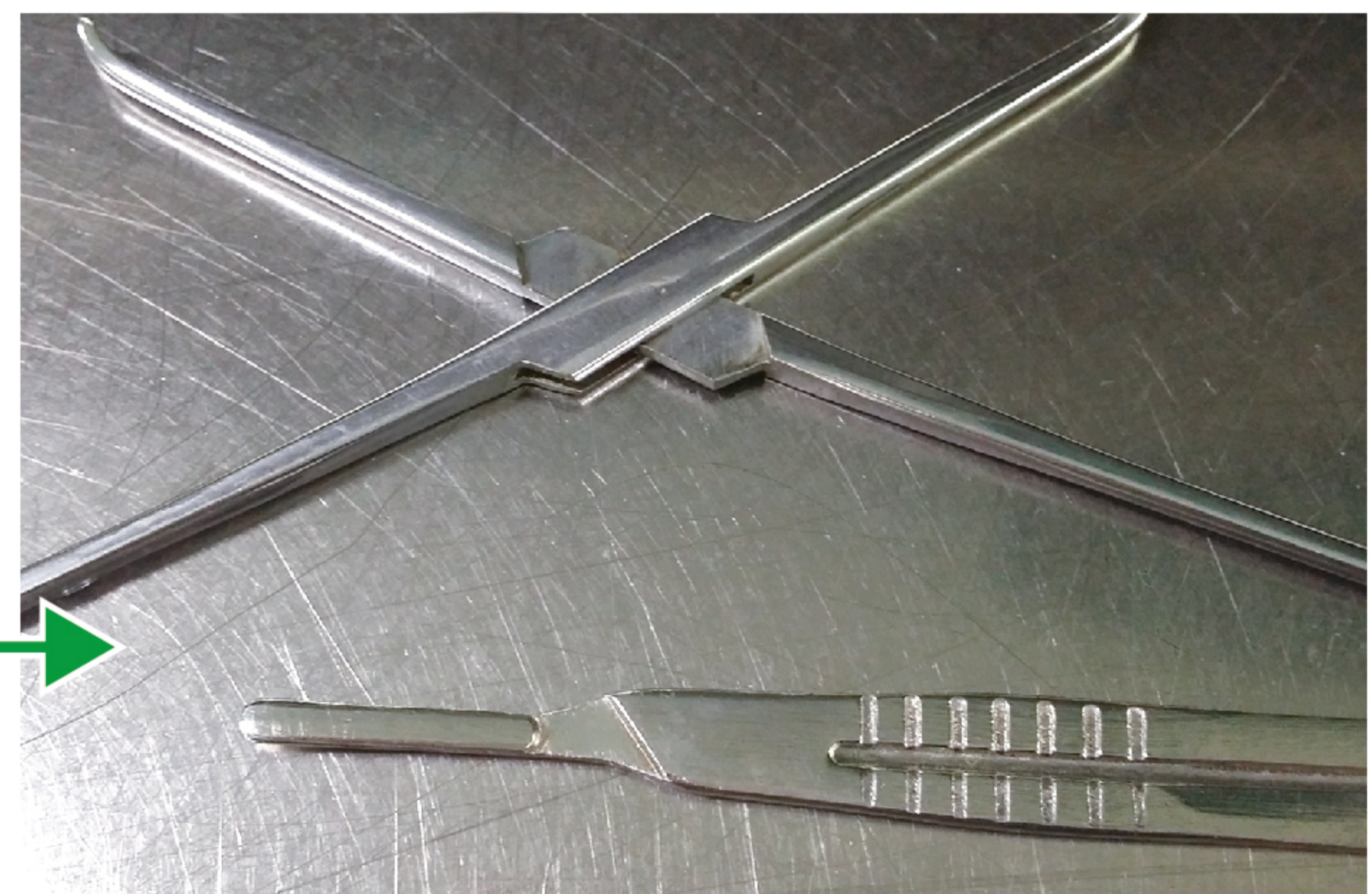

After treatment

Conbizyme® rust remover has a better rust removal effect on the severe rust on the stainless steel (surgical scissors) surface, and the scissor surface can recover the original luster after treatment.

Operating instructions and packing specification

Operating method:

Recommended dilution rate:

[Spraying rust removal]: Conbizyme® rust remover is a ready-to-ues product. It could be sprayed on the rusty sites on average with no dilution. You could scrub it slightly waiting 2-10 min,and then rinsing it until clean. You could also choose the appropriate condition according to the degree of corrosion.

[Soaking rust removal]: Immerse rusted instruments into 1:6-1:10 dilution(add 100mL-166mL as into 1L water) for 5-20 minutes. Select the appropriate conditions according to corrosion situation.

Note: The appropriate rust removal conditions can be selected according to the rust conditions.

Packing specification

Product name	Model	Packing specification
Conbizyme® Rust Remover	3055	500mL (10 bottles per box) 1L (6 bottles per box) 5L (2 bottles per box)

Nanjing Jusha Commercial&Trading Co.,Ltd.

Add: Room 301, No.301, Hanzhongmen Street, Nanjing City, Jiangsu Province, China

Tel: +86/25/8330 5050 Fax: +86/25/5878 3273

Website: www.jusha.com.cn