


InCator[®]

Autoclave Indicator Label 9002

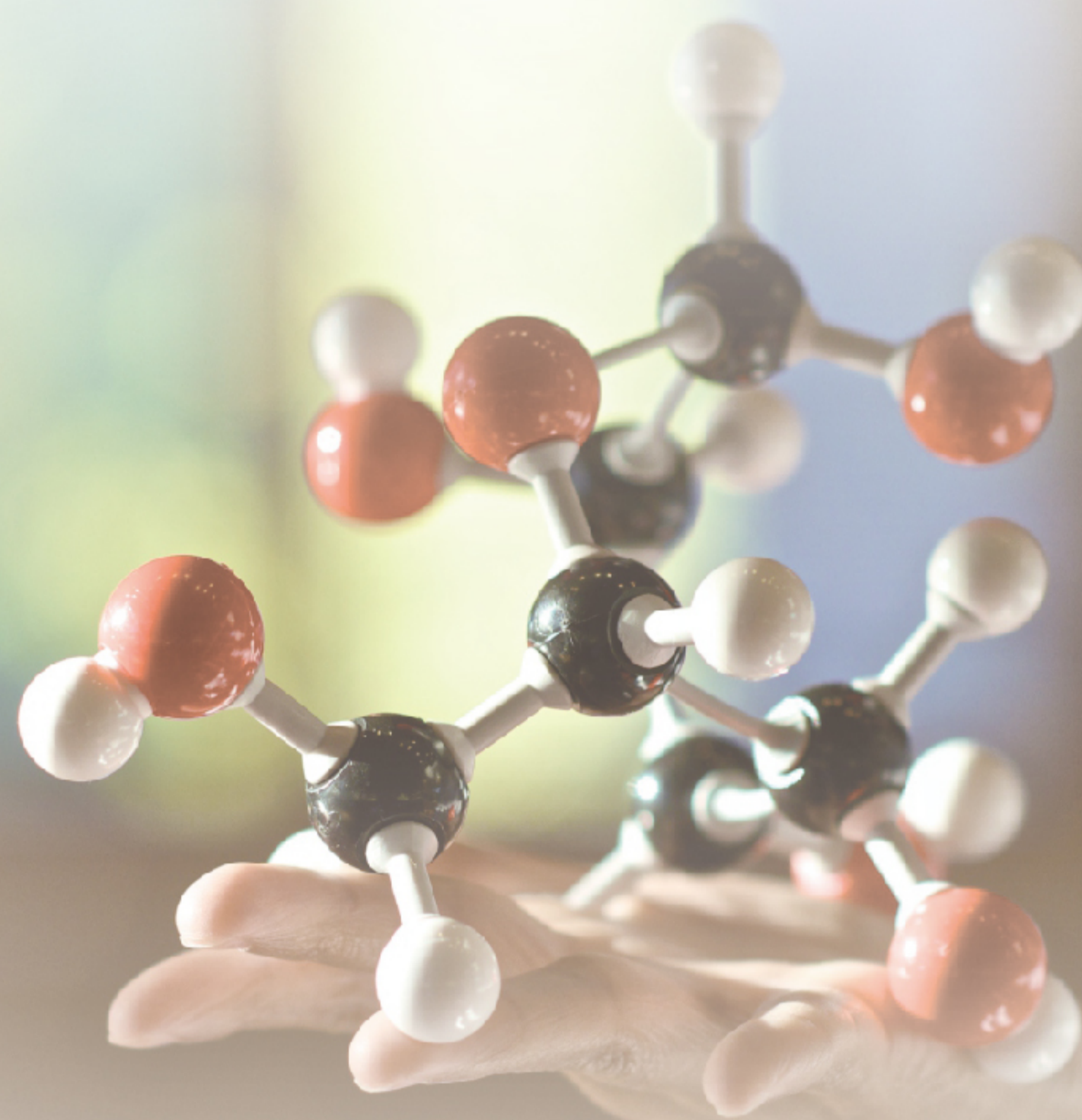


- Customized professional design for easy traceability
- Special glue for steam sterilization, firm and residue-free in removal
- Eco-friendly lead-free formula



Safer Supply, More Than Cleaner

InCator®

Autoclave Indicator Label

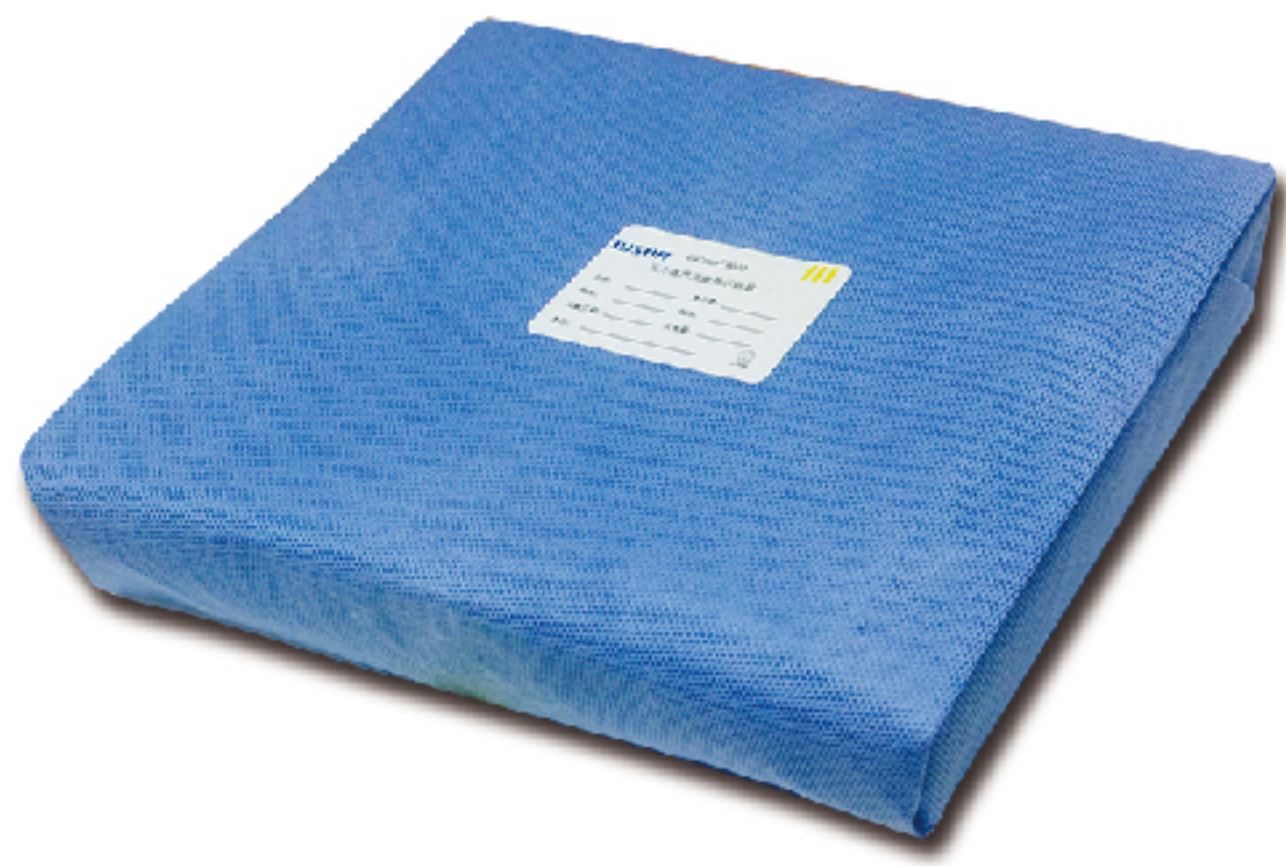

It's bonded to the outside of the medical kit to monitor whether the medical kit is sterilized. The back base adopts the special steam glue and is firm in bonding and free of glue residue. It's suitable for various bonding surfaces, such as cotton cloth, non-woven fabric, crepe paper and metal instrument box. The label can record the sterilizing information and can also be customized as the requirements of customers.

In accordance with Technical Standard for Disinfection and GB18282.1 and ISO11140-1


Technical Standard for Disinfection	132 °C -180s
ISO11140-1,GB18282.1	134 °C -120s

Pass condition


Technical Standard for Disinfection	132 °C -60s 128 °C -180s
ISO11140-1,GB18282.1	134 °C -30s 140 °C -30min(dry heating)

Fail condition


Generous in design and easy to read and discriminate

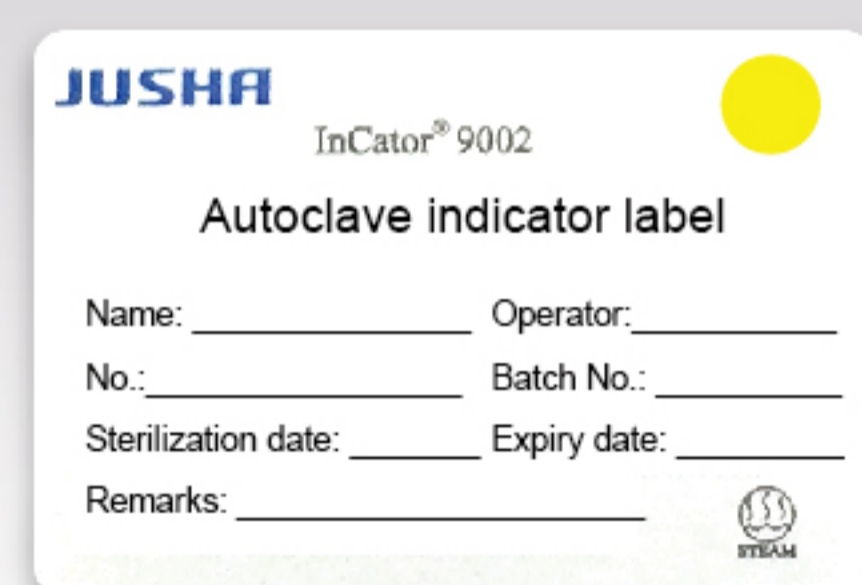

Unprocessed

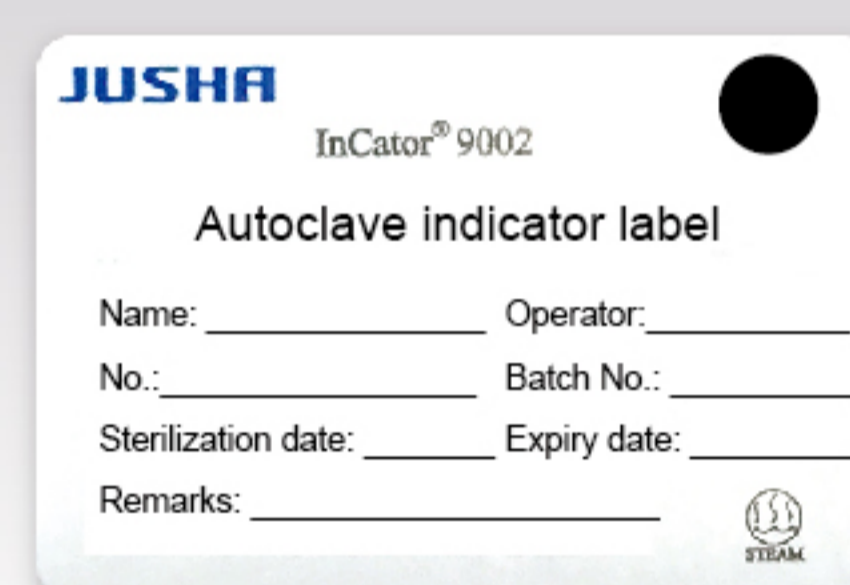


Processed/Pass

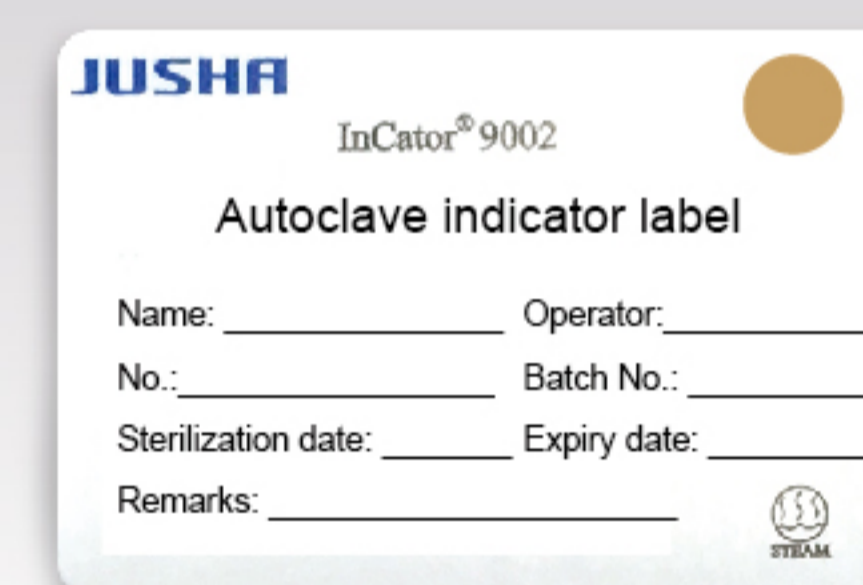

Processed/Fail

Customized special design, convenient to trace

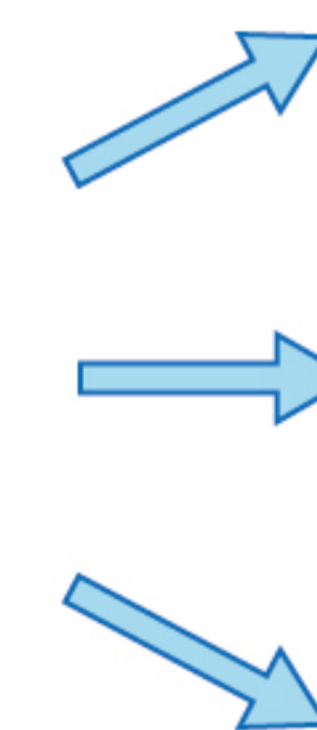

Operating instructions and packing specification

Using instruction:

- 1.Fill out the key information,and then pasted on the outer surface of the pack,peel pouch, unwrapped tray, or rigid container.
2. Run the sterilization process and check the chemical indicator block.
3. After processing, chemical indicator block should have changed into black completely. If indicator block has not changed into black completely or unevenly, a failed result is indicated and the items in the pack, peel pouch, unwrapped tray ,or rigid container were not exposed to sufficient steam sterilization conditions.These items should be returned for reprocessing.

Note:The product should be kept at room temperature (15-30 °C [59-86°F]) with the relative humidity no more than 50% in the cool and ventilation place.

Packing specification

Product name	Model	Packing specification
InCator® 9002 Autoclave indicator label	9002	18rolls/Case

Nanjing Jusha Commercial&Trading Co.,Ltd.

Add: Room 301, No.301, Hanzhongmen Street, Nanjing City, Jiangsu Province, China

Tel: +86/25/8330 5050 Fax: +86/25/5878 3273

Website: www.jusha.com.cn